

PRESTON ANTIMICROBIC SUPPLEMENT

VIAL CONTENTS (for 500 ml of medium)

Cycloheximide	50 mg
Rifampicin	5 mg
Trimethoprim	5 mg
Polymyxin B	2500 IU

DIRECTIONS

Plating Medium: aseptically reconstitute the contents of one vial with 2 ml of 50% acetone solution in distilled water. Prepare 500 ml (or 5 l) of Columbia Blood Agar Base (cat. N° 401136) autoclaved and cooled to 50°C. Add the supplement together with 25 ml of lysed horse blood. Mix well and pour into sterile Petri dishes.

Enrichment Broth: aseptically reconstitute the contents of one vial with 2 ml of 50% acetone solution in distilled water. Prepare 500 ml of Campylobacter Enrichment Broth Base (401286) or 500 ml of Nutrient Broth n° 2 (401812) autoclaved and cooled to 50°C. Add the contents of one vial of Preston Antimicrobial Supplement (code 4240017), together with 25ml of lysed horse blood. If required by internal procedures add the contents of one vial of Campylobacter Growth Supplement (4240021). Mix well and distribute 9 ml into sterile tubes under aseptic conditions.

INTENDED USE

The selective supplement here described is used for the preparation of enrichment media for the isolation of *Campylobacter* in clinical and other specimens. For the details of the procedures see the technical sheets of Campylobacter Enrichment Broth Base (401286) or Nutrient Broth n° 2 (401812).

STORAGE

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date.

PRECAUTIONS AND WARNING

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory. Consult the material safety data sheet before the use

PACKAGING

4240017 Preston Antimicrobial Supplement, 10 vials, each for 500 ml of medium