

## KARMALI ANTIMICROBIC SUPPLEMENT

**VIAL CONTENTS (for 500 ml of medium)**

Cefoperazone 16 mg

Vancomycin 10 ml

**DIRECTIONS**

Aseptically reconstitute the contents of one vial with 5 ml (or 50 ml) of sterile distilled water. Prepare 500 ml (or 5 l) of *Campylobacter* Blood Free Medium Base Karmali (cat. N° 401283) autoclaved and cooled to 50°C and add the supplement. Mix well and pour into sterile Petri dishes.

**INTENDED USE**

The selective supplement here described is used for the preparation of plating medium for the isolation of *Campylobacter* in clinical and other specimens. For the details of the procedures see the *Campylobacter* Blood Free Medium Base (Karmali) technical sheet (401283)

**STORAGE**

Store at 2-8° - When stored as directed the supplement remains stable until the expiry date shown on the label. Do not use beyond stated expiry date.

**PRECAUTIONS AND WARNING**

For *in vitro* diagnostic use only. The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory.

Consult the material safety data sheet before the use.

**PACKAGING**

**4240035 Karmali Antimicrobial Supplement, 10 vials, each for 500 ml of medium**